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## Patient Information Leaflet Vetramil Pad

<div><b>Product</b></div> <div>Vetramil® Pad</div> <div>Moist wound dressing, 10x10 cm</div>
<div><b>Product description</b></div> <div>Vetramil® Pad is a sterile wound dressing consisting of an absorptive layer of natural viscosc which has been coated with a layer of non-adherent polyethylene. The innermost layer is impregnated with a small amount of liquid based on honey and glycerol, as a result of which Vetramil® Pad is slightly moist. Vetramil® Pad contains pure honey that has been produced under controlled conditions. This controlled production process guarantees that Vetramil® Pad contains no residues of pesticides and is always of the same reproducible high quality.</div> <div>The honey used in Vetramil® Pad is rich in honey enzymes, including glucose oxidase.</div> <div>The coating is permeable, so that wound exudate can be absorbed by the inner layer of viscosc. The coating ensures there is no adherence of the dressing to the wound bed.</div>

### How to apply

- Clean the wound, if necessary, in the usual way
- To open the packaging, tear at the notch in the sealed edge
- Remove the gauze from the packaging and apply it evenly over the wound
- Affix the gauze in the usual way, using self-adhesive porous tape or a gauze dressing for example
- Remove the dressing carefully after one to two days (maximum), and replace it with a new Vetramil® Pad dressing

<div><b>Indication</b></div> <div>Vetramil® Pad is suitable for covering superficial wounds</div>
<div><b>Contraindications</b></div> <div>1. Do not use in the event of hypersensitivity to honey</div> <div>2. If any unexpected (or unrecorded) side-effects occur after use of Vetramil® Pad, please contact both the person providing treatment and the manufacturer</div>

<div><b>Precautions</b></div> <div> <ul style="list-style-type: none"><li>Do not use if the packaging is damaged</li> <li>Do not use if the packaging is not properly sealed</li> <li>Do not treat a wound with the product for more than 30 consecutive days</li> <li>Do not re-use a used dressing</li> <li>Do not resterilise an opened package</li></ul></div>
<div><b>Storage conditions</b></div> <div>Store between 4 and 30°C</div>

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<div><b>Contraindications</b></div> <div>1. Do not use in the event of hypersensitivity to honey</div> <div>2. If any unexpected (or unrecorded) side-effects occur after use of Vetramil® Pad, please contact both the person providing treatment and the manufacturer</div>
<div><b>Precautions</b></div> <div> <ul style="list-style-type: none"><li>Do not use if the packaging is damaged</li> <li>Do not use if the packaging is not properly sealed</li> <li>Do not treat a wound with the product for more than 30 consecutive days</li> <li>Do not re-use a used dressing</li> <li>Do not resterilise an opened package</li></ul></div>
<div><b>Storage conditions</b></div> <div>Store between 4 and 30°C</div>

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